
# bk3500 Quick Start Guide

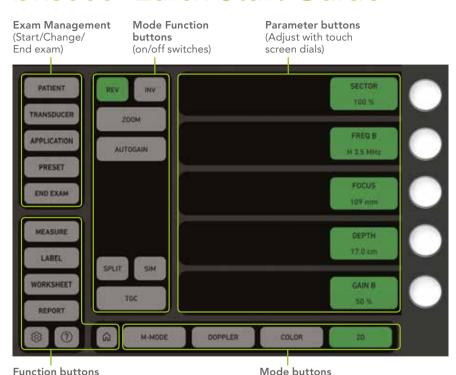

Basic/Advanced


(Measure/Label/Report/Settings/Help/Home)

You can switch touch screen layout between Basic and Advanced in **Settings**This is Basic layout.



The power button is located top left on the scanning engine.

(Switch imaging mode)

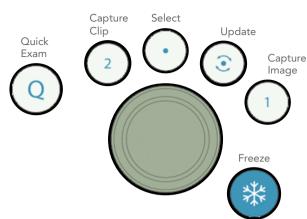

#### Begin an Exam

- 1. Press
- 2. Enter **Patient Information** and tap **Next**.
- 3. Select a **Transducer** connected to the system.
- 4. Select **Application** based on your transducer selection.
- 5. Select Preset.
- 6. Start the exam.







## bk3500 Functions

#### Measurements

- to get measurement calipers.
- Measurements specific to the active application are available. A measurement package can be changed in the drop down menu.

## **Capture Images and Clips**

- to capture a still image.
- Press (2) to capture a clip.
  - a. on the live image to store a prospective capture.
  - b. on the frozen image to store a retrospective capture.

#### Label an Exam





Select the **Bodymarks** tab for bodymarks.

## **Review Images and Clips**



- Doubleclick an image thumbnail on the clinical display to review an acquired image or clip.
- Tap **CLOSE** on the touch screen to close the image or clip review.

## **Using Worksheets**

to fill in a worksheet for use in your report.

#### **End an Exam**

Tap the touch screen button.

## **Create a Report**

to review, edit, save and print a report.



BOOVWARKS

Lang Aris