

Navigating the Touchscreen



Tap

- Any button to select or deselect
- Trackpad to lock measurement calipers



Double Tap

- 2D to exit all other modes
- An exam on the Patient List to review



Swipe

- Gain and Cine bar; side to side
- Depth and Focus bar; flick up or down



Tap + or - for +/- single unit steps

- Gain bar
- Cine bar



Slide

- TGC
- Trackpad when measuring



Tap and Slide

- Tap Color; tap and slide trackpad to adjust ROI box size. Tap to set size
- Slide trackpad to move ROI box

Power Button

You can turn the system on or off using the power button located on the monitor or on the scanning engine (top-left corner).



Exam Management

Transducer

Current User and Help

Settings

Input Area

Basic Touchscreen Buttons

Begin an Exam

- 1 Enter patient details and tap **NEXT**
- 2 Select a transducer connected to the system
- 3 Select Exam Type and Preset
- 4 Tap **START EXAM**


You can also start the exam immediately after entering patient details. Tap **START EXAM**. Select transducer using the transducer control button. The system will use the default exam type and preset.

1 Begin an Exam

Follow Steps 1-4 on page 1.


2 Color Mode

Tap **COL** to activate. Tap **POWER DOPPLER** to toggle Power Doppler on/off.

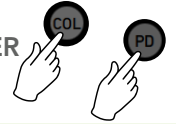


3 PW Doppler Mode

Tap **PW** to activate. Tap **TRIPLEX** to toggle Triplex off/on.

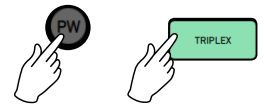

4 Measure and Calculate


Tap **Freeze**.* Tap **CALC** for calculations. Tap **MEASURE** for Generic measurements. Use tap and slide to position calipers.

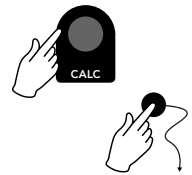

*CALC can be auto-selected on FREEZE

5 Label an Exam

1. Tap **LABEL**. Then tap desired entry type:

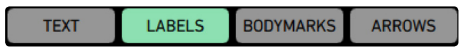


6 Capture

Tap **STORE** to capture still frame. Or tap **Freeze**, select **Cine Frame**, then tap **STORE**. Tap **CLIP** to capture a clip.



7 Needle Guide for Biopsy

Tap **BIOPSY** button to activate/deactivate.

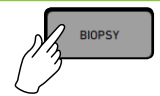

8 Review Exam

1. Tap **REVIEW**. Current exam images will appear.
2. If no exam is open then double-tap on desired exam from the list.




9 Create Report

Tap **REPORT** to create, preview, print and store a report. Images are automatically added to the report.




10 Pause an Exam

Tap **PAUSE EXAM**.

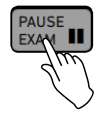

10 End an Exam

Tap **END EXAM**.

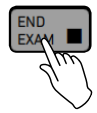

Helpful Image Adjustments

Gain

Tap -/+ or slide on the Gain bar to adjust.



Depth/Focus

Flick up or down on the **Depth/Focus** bar to adjust.



Home

Tap home to bring image back to default settings.



Needle Enhance

Locate and tap Needle Enhance. Select **RIGHT** or **LEFT** to indicate needle entry point.